

# North American Aluminum Crutches

Exclusively offered by




## Available Now




High quality, 100% North American made, **Adjustable Crutches** constructed from 6061-T6 aerospace grade anodized aluminum.

- Competitive prices
- Manufactured in a Class 1 Medical Device facility in North America
- Engineered for greater weight bearing and durability
- Innovative T joint allowing for maximum capacity and stability
- Choice of 3 aluminum colors:

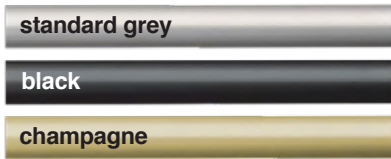

- ✓ 14 points of adjustment
- ✓ No tools necessary

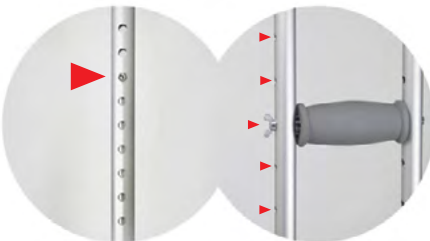

5 handgrip adjustment points

9 height adjustments with stainless steel push button



Manufactured by G-Force Medical

Hawktree Solutions is G-Force's exclusive global master distribution partner



### ADJUSTABLE ALUMINUM CRUTCHES

Latex Free • 6061T Anodized Aluminum • 350lb capacity

#### Size

- GFCA-S 4' - 4'6" (122cm - 137cm)
- GFCA-M 4'6" - 5'2" (137cm - 157.5cm)
- GFCA-L 5'2" - 5'10" (157.5cm - 178cm)
- GFCA-XL 5'10" - 6'6" (178cm - 198cm)

#### Instructions

The patient should consult a trained health professional for an exact adjustment and to be instructed on proper walking pattern.

- The top of the crutches should be between 1.5 to 2 inches below patient's armpit while standing up straight.
- Adjust crutches to the correct height using the adjustment push-button.
- The handgrips of the crutches should be even with the top of patient's hips.
- Adjust handgrips to the correct height. There should be a 20-degree bend in the elbow when patient is using the handgrips.

#### Cautions & Warnings

- Do not press the top of the crutches into armpits. This could result in damage to the nerves that run under the patient's arms and possibly loss of balance.
- Periodically inspect handgrip bolts and wing-nuts tightness. The handgrips should be firmly held in place by the bolt and wing-nut. Stop using the crutches and immediately inspect bolts and wing-nuts tightness if feeling any movement in the handgrips. If damage is noticed, replace handgrips assembly. (SKU: GFCA-HGG-1)
- Periodically inspect crutch tips. They must be securely attached, free of cracks and damages. Stop using the crutches and immediately replace crutch tips if they become loose, if there is any visible damage or if slippage is observed. (SKU: GFCA-TIP78-01)
- Employ extra caution on wet or slippery surfaces.



Distributed by

Hawktree Solutions



# Adjustable Aluminum Crutches made in North America

Domestic production ensures a stable supply chain, eliminating logistic delays and product shortages.

## GFCA-S

Client Height Range:  
4' - 4'6"

## GFCA-L

Client Height Range:  
5'2" - 5'10"

## GFCA-M

Client Height Range:  
4'6" - 5'2"

## GFCA-XL

Client Height Range:  
5'10" - 6'6"

Unit	Dimensions (in.)	Weight (lbs)
Each (Pair of 2)	-	3
Case (10 Eaches)	54 x 19 x 21	30
Pallet (8 Cases)	54 x 87 x 38	240

Size	Each SKU	Each UPC	Case SKU	Case UPC
Small	GFC-A-S	695956003023	GFC-A-S-CS	00695956003047
Medium	GFC-A-M	695956002934	GFC-A-M-CS	00695956002958
Large	GFC-A-L	695956002965	GFC-A-L-CS	00695956002989
Extra Large	GFC-A-XL	695956002996	GFC-A-XL-CS	00695956003016

For more information on availability or pricing, please reach out to us.



**KwikTemp Medical**  
 UEI: HAC5GD8G44A6 • CAGE Code: 8MC97  
 246 North New Hope Road, Gastonia, NC 28054  
 Phone: 980-253-0949 • Contact KwikTemp  
[www.kwiktempmedical.com](http://www.kwiktempmedical.com)

